

Biogal's ImmunoComb® VacciCheck

Canine Distemper - Adenovirus Type 2 - Parvovirus Antibody Test Kit, Dot Blot

The Canine VacciCheck is a quick, simple and portable dot blot test kit intended for detection of antibodies to:

- Infectious Hepatitis (tested by Adenovirus Type 2 - CAV2)
- Parvovirus (CPV)
- Distemper (CDV)

What benefits can the Canine VacciCheck provide?




Canine VacciCheck can:

- Assist in diagnosis of disease status of dogs from one of these diseases.
- Help determine if a dog requires additional vaccination, thus reducing unnecessary vaccination and potential side effects.
- Be useful to determine a dog's unknown vaccination history.
- Determine if puppies have received immunity from initial vaccination program.

How does the Canine VacciCheck provide such benefits?

The Canine VacciCheck:


- Provides semi quantitative serological results for the three core vaccines.
- Checks these three vaccines in one test in the veterinary clinic or lab.
- Supplies results within 21 minutes.
- Does not require a lab or lab technician to perform the test.

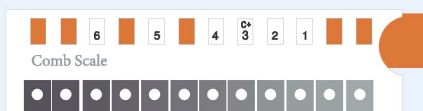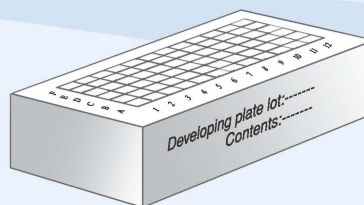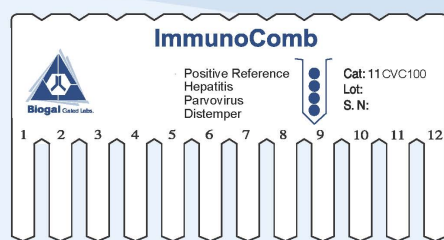


Canine Distemper - Adenovirus Type 2 - Parvovirus Antibody Test Kit, Dot Blot

Two main components:

The **developing plate** has multiple compartments, which contain the necessary reagents for Comb development.

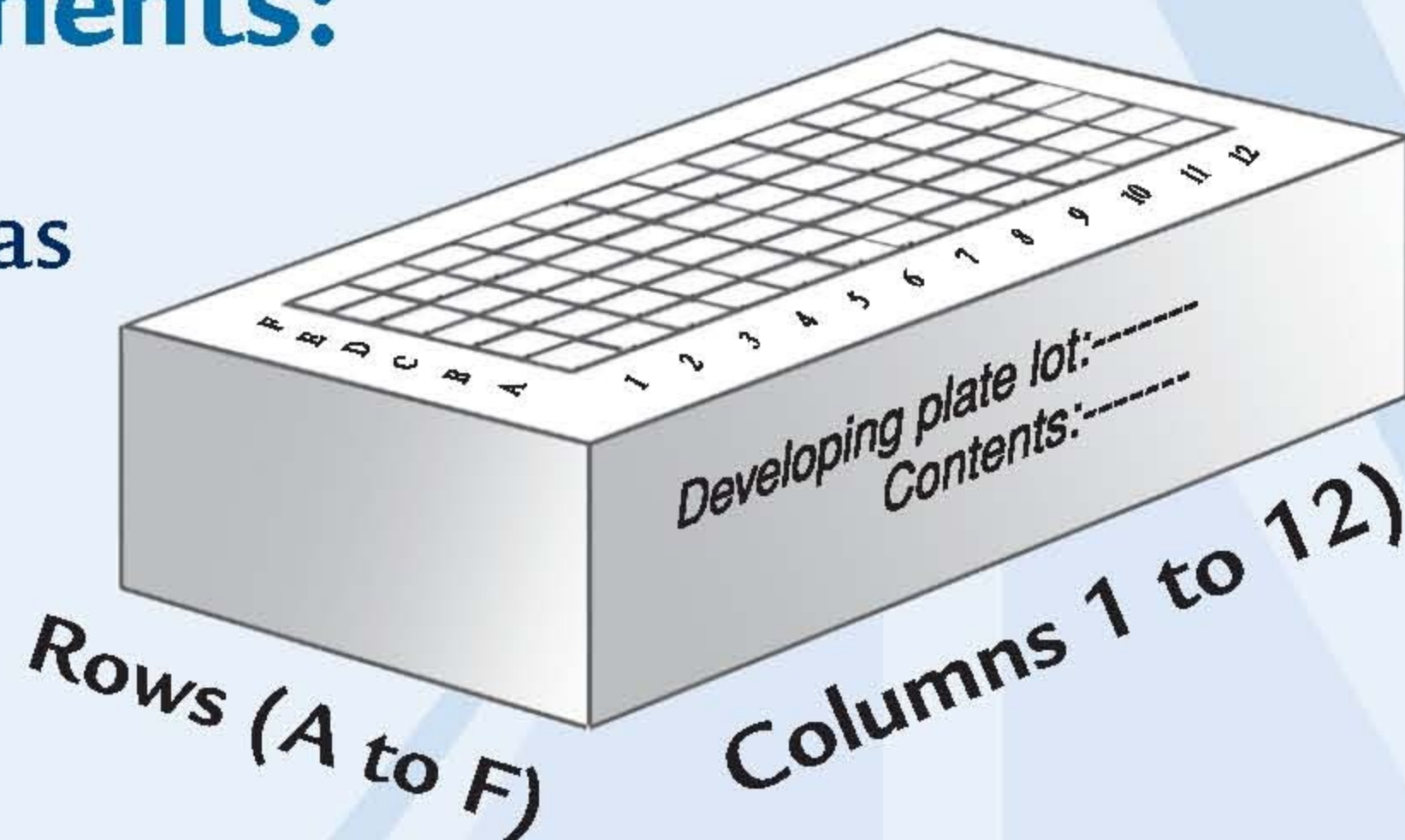

- The **Comb** has 12 teeth.
- One tooth tests one blood/serum sample.
- Each tooth can provide up to three tests: CAV2, CPD and CDV

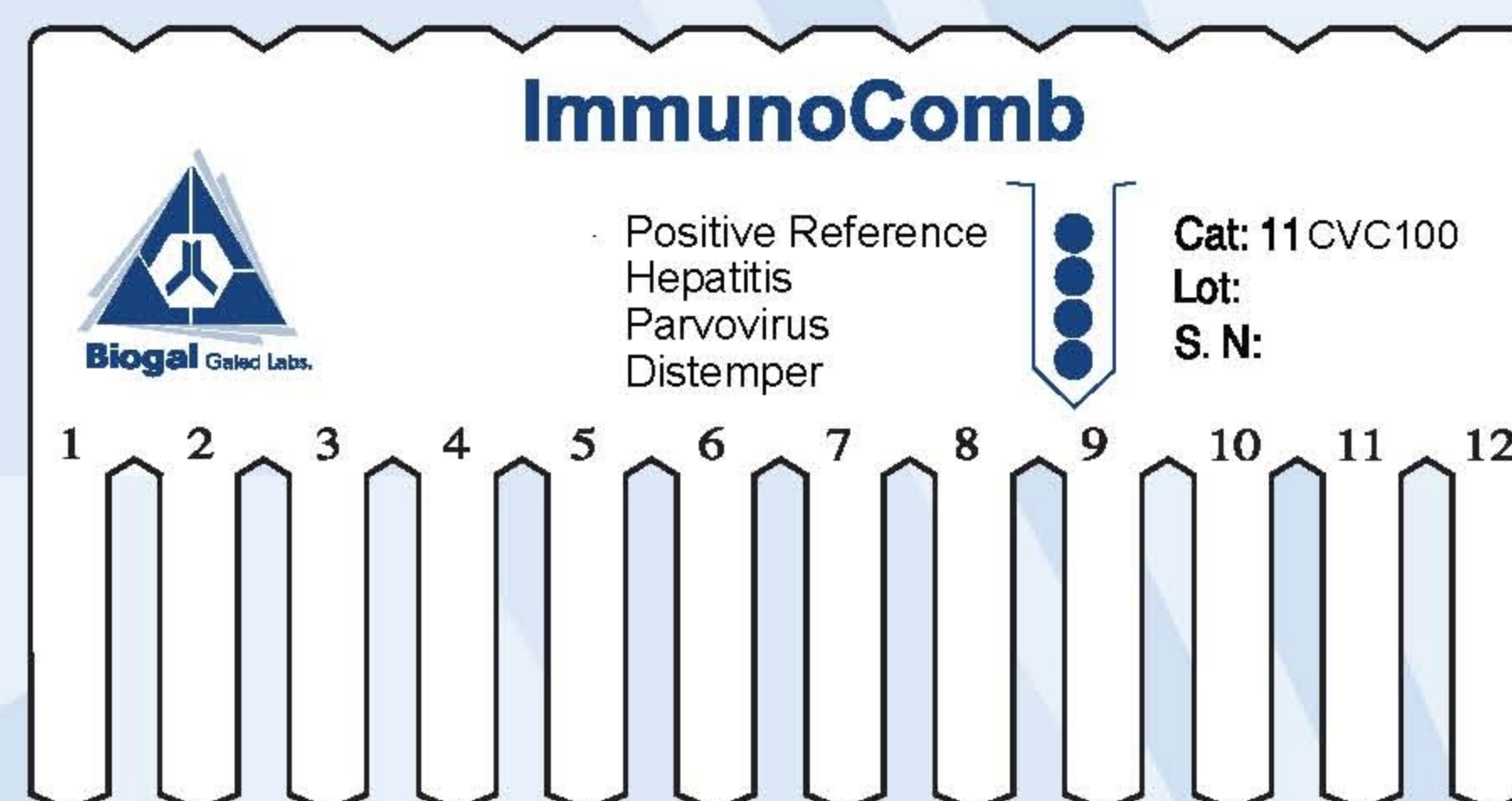

Kit Development Process:




1 Obtain blood from animal(s).




2 Deposit blood or serum sample(s) into well(s) A in developing plate.



3 Insert Comb into well(s) A. Incubate for a specified time.

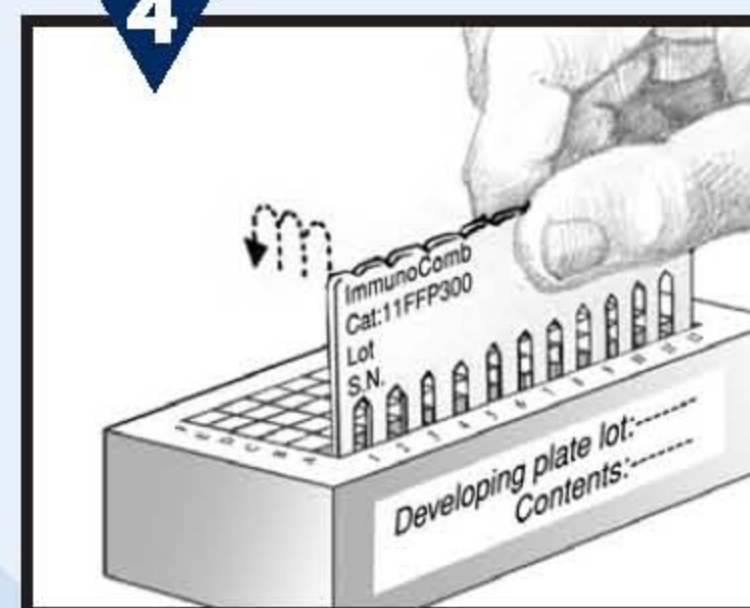

4 Transfer the Comb from well(s) A to well(s) F in a timed sequence. Time intervals vary according to type of Kit.

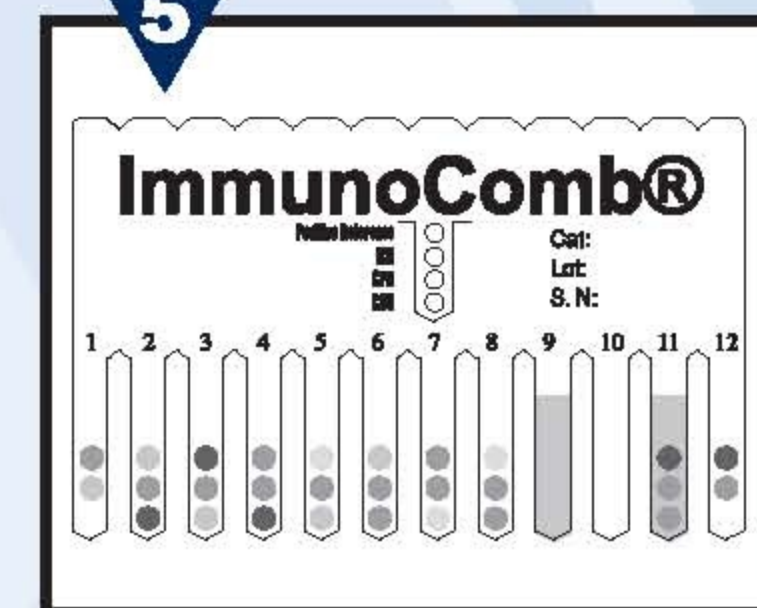

5 Let Comb dry and read results. The color intensity of spots on Comb corresponds to antibody level.

For Further Information Contact:

In the USA:

Modern Veterinary Therapeutics, LLC
18001 Old Cutler Road, Suite 317 - Miami, Florida - 33157
Tel: (305) 234-8228 Fax: (305) 503-8585
e-mail: info@modernveterinarytherapeutics.com
Website: www.modernveterinarytherapeutics.com