# Rapide FCoV Ab Test Kit

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#### Modern Veterinary Therapeutics SAS

5, Chemin du Champ Boisnel  
35800 Saint-Lunaire - FRANCE

Tel. +33 (0)2 99 46 30 77

Fax +33 (0)2 22 44 19 26

info@modernveterinarytherapeutics.com

www.modernveterinarytherapeutics.com

# Rapide FCoV Ab Test Kit

## Rapid Feline Coronavirus Antibody Detection Test

It is estimated that up to 70% of cats, worldwide, are exposed to Feline Enteric Corona Viruses (FCoV). Infection is transmitted by the fecal-oral route. In most cats, infection with FCoV is asymptomatic. In some cats, however, a severe systemic disease known as Feline Infectious Peritonitis (FIP) may develop. Clinical FIP has 2 forms: effusive (wet) and non-effusive (dry). FIP is generally associated with a fatal outcome, even with therapy.

### Indications:

- A negative result is helpful in ruling out a diagnosis for FIP
- A positive result means that the cat has previously been exposed to FCoV and potentially FIP. A definitive diagnosis of FIP should be based only on complete clinical and laboratory findings
- Other indications: Testing before mating; screening a household or cattery for the presence of FCoV; screening a cat before introduction into a FCoV-free household or cattery.

### Principles:

Rapid chromatographic immunoassay for the qualitative detection of FCoV antibodies in feline **whole blood, plasma or serum**. Results within **10 minutes**.

- (1) 1 drop of specimen (2) 3 drops of buffer solution (3) Results in 10 min

### Performance:

In a study using 149 samples, **Rapide** - FCoV Ab Test kit was compared to Immuno-Fluorescence Antibody Test (IFA), a reference standard technique in laboratory diagnostics, the **sensitivity was 96%** and **specificity was 97.9%**. In another field trial study taking place at multiple veterinary clinics in the United States, the **sensitivity and the specificity were 100%** compared to IFA.

### Packaging sizes and storage:

10, 20, 30, 40, 50 tests per kit; store at room temperature

#### Modern Veterinary Therapeutics SAS

5, Chemin du Champ Boisnel  
35800 Saint-Lunaire - FRANCE

Tel. +33 (0)2 99 46 30 77

Fax +33 (0)2 22 44 19 26

info@modernveterinarytherapeutics.com

www.modernveterinarytherapeutics.com